

# Torque guide

## Recommended tightening torque

| Type of product   | Image | Torque – Ncm |          |          |
|---|-------|--------------|----------|----------|
|   |       | X-Small      | Small    | Large    |
| Cover Screw   |       | Manual*      | Manual*  | Manual*  |
| Healing Abutment<br>Healing Abutment Uni<br>ProHeal Cap<br>Healing Cap Angled                                     |       | Manual**     | Manual** | Manual** |
| TempDesign™<br>Temporary Abutment   |       | -            | 15       | 15       |
| 20°/45° Cresco™ Insert for<br>Astra Tech Implant System™<br>20°/45° UniAbutment                                   |       | -            | 15       | 15       |
| Bridge Screws<br>Cresco™ Bridge Screw   |       | -            | 15       | 15       |
| Atlantis™ abutment for<br>Astra Tech Implant System™<br>ZirDesign™<br>TiDesign™<br>CastDesign™<br>Angled Abutment |       | 15***        | 20       | 25       |
| Direct Abutment™<br>Ball Abutment<br>Locator™ Abutment  |       | -            | 25       | 25       |

\* Only light finger force (5–10 Ncm) using a manual screwdriver or contra angle preset at 25 rpm and 5–10 Ncm torque.

\*\* Only light finger force (5–10 Ncm) using a manual screwdriver. Do not use a Ratchet Wrench or Torque Wrench.

\*\*\* **Note:** Available for TiDesign™, Atlantis™ abutment in titanium and Atlantis GoldHue™ abutment.